■ DENCRILAY PATTERN

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PÓ - Cadastro ANVISA N° 10216040031

■ DENCRILAY PATTERN

Autopolymerizable acrylic resin for sculpting casting patterns.

Powder composition:

- Poly methyl methacrylate
- Benzoyl Peroxide
- Biocompatible Pigments

Liquid composition:

- Methyl Methacrylate
- DMT
- Inhibitor
- EDMA
- Fluorescent

Indication:

Dencrilay is used by both the prosthetist and the dentist, who make intraradicular moldings to make molten cores, sculptures of fixed prostheses and the fixation of their soldering tips, in addition to making mold headgears, fixation of implant copings and all works where precision is required.

Colors:

Red

Packaging:

Kit: 50g Resin, 50 ml Liquid and accessories.

Resin: Bottle of 25, 50 and 80g.

Expiry:

- **Liquid:** 2 years as from the date of manufacture.
- **Powder:** 10 years as from the date of manufacture.

Instructions for use:

- To make the molding of the intraradicular canal, it is important to use a guide pin made in Dencrilay. The canal should be insulated with insulator (provided with the Kit), put the powder and liquid each in a Dappen pot (indirect technique), with the use of a Marta brush (provided with the Kit), fill the canal with the resin, wetting the brush in liquid and later in the powder and introducing the guide pin.
- To make the sculptures and headgears, insulate the plaster model with Isocril insulator, and then proceed with the indirect technique described above or with the direct technique.
- Direct technique:
 - 1- Put the liquid inside a Dappen pot and then the powder in the ratio of 2:1.
 - 2- Spatulate as minimum as possible, cover the Dappen until the plastic phase is reached (phase when the resin breaks without forming threads).
 - 3- When the plastic phase is reached the resin can be transferred:
 - to the plaster model (insulated with Isocril)
 - to the patient's mouth.

Benefits of the product:

- Researches conducted by UNESP show that Dencrilay has negligible contraction (0.09%) and does not leave residues after the inclusion of the resin pattern on the coating.
- Color: red
- The strict selection and receiving tests of the raw materials combined with the successive tests during all the manufacturing cycle, under the ISO requirements, ensure the specified results in the final product.
- Excellent dimensional stability;
- Unmatched brilliance;
- Easy to handle;
- Easy application;
- Provides excellent cost and benefit, considering the precision in the molding;
- Economic packaging.

DENCRILAY is a product easy to handle and apply, with fast and accurate results.

Competitive edge:

- Selected raw materials.
- Strict manufacturing controls.
- Easy to handle and apply to perform the work.
- Practical packaging.
- Best cost benefit;

Frequently asked questions:

Why does the liquid polymerize?

The liquid polymerizes when exposed to sunlight or sources of heat (ovens and heaters). To avoid this from happening, it is recommended to store it in a fresh and dark place.

What are the causes of a slow polymerization?

The temperature is a direct influence in this process. A slow polymerization is caused by low room temperature. However, if the temperature is high, the process is accelerated.

How can I increase the handling time of the product?

In order to have more handling time of the product, it is recommended to use the liquid in the refrigerator before use on very hot days.

Precautions:

- Only for dental use.
- Flammable material, non toxic.
- It is recommended to handle the product in ventilated places, using nitrile gloves and protection goggles; however, handling is not recommended for pregnant women, nursing mothers and people with known allergies to this type of product.
- It can cause skin irritation in susceptible people.
- Do not dispose the wastes in sewers and water bodies or in domestic waste.
- Products that are not compatible with the resin/monomer: alcohol, water, chloroform, acetone, vaseline and other solvents.
- Store away from sunlight and heat, closed in a dry and cool place (between 2°C and 28°C).

Note:

Products that are not compatible with the resin/monomer: alcohol, water, chloroform, acetone, vaseline and other organic solvents.

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